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ART. 7035

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Description de l'article:

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ANKLE SUPPORT ELASTIC

Product description:

The bandage is manufactured by knitting technology from breathable material with high content of natural cotton. It provides moderate fixation, relieves pain and hastens recovery. It remains flexible for a long time ensuring good compression of the joint.

Indications for use:

Trauma of the ankle joint. Ligamentous laxity of the ankle joint. Chronic arthritis and initial stages of arthrosis. Late rehabilitation after a plaster cast is removed. Prevention of habitual dislocation or trauma caused by sport activities.

- The anatomically shaped bandage fits well for everyday usage.
- Breathable material ensures maximal comfort.
- The bandage fits both left and right legs.

Recommendations for use:

It is obligatory to consult your doctor before use. Size of the bandage is chosen according to the ankle circumference measurements as shown in the Table. It is recommended to put the bandage on in the sitting position when your foot is at 90 degrees to your leg. The bandage regimen and duration of wearing are recommended on an individual basis by a medical doctor.

Contraindications:

Inflammatory skin disease in the area of brace application, individual intolerance to the brace materials.

Dimensions	Sizes				Max deviation
	S	M	L	XL	
Ankle-joint circumference, cm	15-20	21-26	27-32	33-38	± 1,0 cm

**Maintenance:**

Wash with mild detergents at temperature no higher +35°C. Don't rub or wring out during washing and rinsing. Dry up away from sources of heat. Don't iron. Don't bleach. (Don't use chlorine-containing bleaching detergents). Don't use dry-cleaning.

Precautions:

Use the product exceptionally by prescription. Regulation and changing of the degree of support prop must be implemented only according to your doctor's recommendations. It is not recommended to put the bandage onto the ankle within the first 30 minutes after applying hot ointment.

Contents:

cotton - 75%, latex - 17%, polyether - 8%.

Storage conditions:

Store in dry place at temperature from +5°C to +40°C. Keep away from direct sun rays.

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MedPack Swiss GmbH ist ein Mitglied der MedPack Swiss Group, Schweiz.