



THANK YOU FOR YOUR INTEREST IN *WHEELCHAIRHANDLES*

READ DISCLAIMER BEFORE USE.

WHEELCHAIRHANDLES ARE TO BE USED FOR INTENDED PURPOSES ONLY. USER ACCEPTS ALL RESPONSIBILITY TO ENSURE *WHEELCHAIRHANDLES* ARE TIGHT AND SECURED PROPERLY BEFORE EACH USE.

INSTRUCTIONS

- 1) Make sure existing vinyl/plastic wheelchair grip is affixed securely to wheelchair. Do not use if existing vinyl/plastic wheelchair grip is loose. (If safe, remove damaged grip and attach *Wheelchairhandle*.)
- 2) Place *Wheelchairhandle* over vinyl/plastic grip.
- 3) Make sure that the end of the wheelchair handle penetrates completely through the *Wheelchairhandle*.
- 4) Align *Wheelchairhandle* to preference (recommended with top portion straight up).
- 5) Tighten bottom screw.
- 6) Take turns alternating side screws several twists until each side is tight.
- 7) Grab top of *Wheelchairhandle* and wiggle for looseness.
- 8) Re-adjust all three screws for tightness.
- 9) Repeat until *Wheelchairhandle* is firmly secured.
- 10) *Wheelchairhandles* should be tight and unable to be twisted around. (If tightness cannot be achieved, do not use)
- 11) Check for tightness before each usage. If loose, retighten.
- 12) To remove *Wheelchairhandles* later, loose the screws.



! WARNING / CAUTION / NOTICE !

THESE MEDICAL DEVICE GRIPS SHOULD BE USED ONLY AS INSTRUCTED. EXERCISE EXTREME CAUTION AND CARE WHEN USING THE GRIPS TO ASSIST WHEELCHAIR TRANSPORT. THE GRIPS ARE MADE TO BE USED ONLY FOR WHEELCHAIR TRANSPORT ASSISTANCE. DO NOT USE THE GRIPS FOR ANY OTHER PURPOSE. YOU AND THE WHEELCHAIR PASSENGER MUST READ AND UNDERSTAND THE INSTRUCTIONS ACCOMPANYING THE GRIPS BEFORE INSTALLING AND USING THE GRIPS. IMPROPER USE OF THE GRIPS MAY RESULT IN SERIOUS BODILY INJURY TO BOTH THE INDIVIDUAL IN THE WHEELCHAIR, THE INDIVIDUAL PUSHING THE WHEELCHAIR, AND OTHERS WHO ARE IN THE SAME GENERAL VICINITY OF THE WHEELCHAIR.