forte rhinoBeam

D
GB
F
I
E

Model RB-01
Version: 1.1

forte rhinoBeam

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GB Congratulations!

We congratulate you on your purchase of a rhinoBeam therapy device, a modern microprocessor controlled, optical therapy device that can be used to effectively maintain and increase your bodily well-being.

Please carefully read the instructions for use and above all take note of the safety instructions before using your rhinoBeam therapy device.

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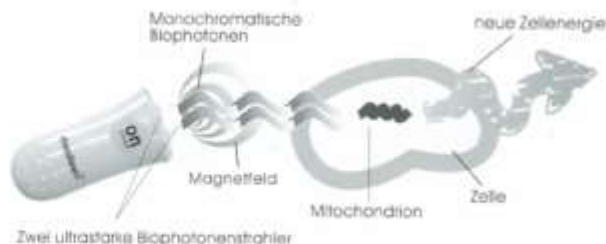
GB

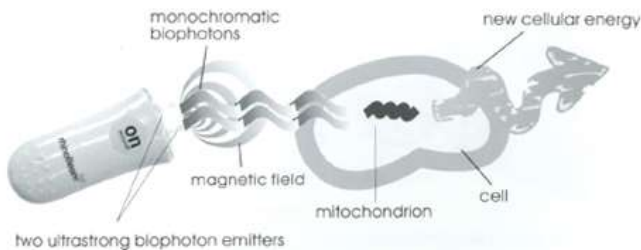
How rhinoBeam forte works.

The rhinoBeam therapy device emits monochromatic light waves that penetrate deep into body tissue. RhinoBeam is capable of remedying a cellular energy crisis because the ultrastrong photon emitters act directly on the cells. This treatment leads to the production of ATP (adenosine triphosphate), a molecular-level cellular energy. This can lead to a noticeable improvement in an ill-health condition. During the course of a rhinoBeam treatment, the patient also undergoes magnetic field stimulation. This can result in a boosting of the body's natural resistance.

We recommend that the rhinoBeam treatment be undertaken in a relaxed seated or lying position. To increase your immune resistance, it is advisable to carry out the treatment two to three times daily (5 minutes at a time).

If despite using the device correctly the symptoms you are treating using rhinoBeam have still not eased after three to four days, or you cannot discern any tendency towards improvement, you should seek medical advice.

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GB

Important instructions!

- * Carefully read all instructions.
- * Never look directly into the rhinoBeam light beam, even using an optical instrument (magnifying glass, etc.)
- * If any aspect of usage is unclear, we recommend that you consult your doctor.
- * Keep the device out of reach of children.
- * The device must not be used by persons wearing a pacemaker!
- * Avoid operating the device in the vicinity of anesthetic gases or flammable substances.
- * Remove batteries if you won't be using the device for a longer period of time. Replace the batteries after approx. 200 treatments.
- * Repairs should only be carried out by those authorized service agents whose address has been specified here.

Cleaning

Sterilise light conductors in boiling water for 10 minutes prior to each treatment. Use only a moist cloth to wipe the rhinoBeam device.

Technical specifications:

Device type:	Light emitting diodes class 1	
Protection class:	internal supply	
Wavelength:	620-660 nm	
Output:	max. 5 mW	
Power supply:	2 x 1,5 V alkali-battery, AA, LR6	
Nominal voltage:	3 VDC (Min. 2.2v)	
Measurement:	117 x 50 x 25 mm	

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GB Using the rhinoBeam forte

Switch on the device and carefully insert the light emitters into the nose. Breath through the mouth during the treatment. The device will switch itself off automatically after 5 minutes.

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GB Getting started

1. Connect the light emitters "A"
2. Open the battery cartridge "B"
3. Insert batteries "C"

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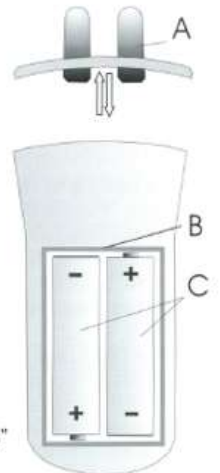
forte

Warranty GB

We grant a 2 year guarantee on our product commencing on the date of purchase.

Within this guarantee period we shall settle free of charge all defects found to be due to faulty material or workmanship, either by repair, exchange of parts or whole unit.

This guarantee does not apply to damages brought about through improper handling (wrong electric current/voltage, connection to unsuitable mains supply, breach), normal wear, or defects which affect the usefulness of the unit insignificantly.

The guarantee expires in cases of intervention by unauthorized parties, or by use of not original spare parts. The guarantee becomes effective only, if the date of purchase is confirmed on the guarantee card and the registration card with stamp and signature of the dealer.

This guarantee is valid in all countries where this unit is sold by authorized dealers and in which restrictions of import or other legal regulations do not oppose against it.

In the event of a claim the complete unit, accompanied by the correctly filled in guarantee card, is to be sent to our authorized dealer, or both sent to the next customer service representative. Any other claims of any kind are excluded from this guarantee, unless our liability is not imperatively prescribed. This guarantee claims of the contract of purchase against the seller are not affected by this guarantee.

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RHINOBEAM AG - CH-6340 Baar - Switzerland
For technical documents please contact Rhinobeam AG