


Assembly


1/ Match the stimulator and integrated cuff (right stimulator for right integrated cuff and vice versa)



2/ Insert the stimulator into the integrated cuff in the correct direction

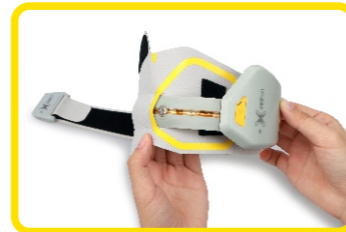

3/ Attach the Velcro

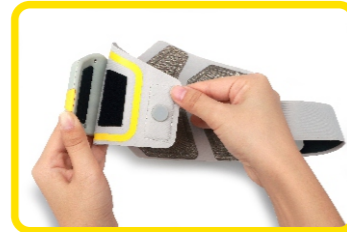

4/ Use finger to feel the place of the snap buttons and fasten them



QUICK START GUIDE


Assembly and Disassembly of the Integrated Cuff




www.xft-china.com

Disassembly


1/ Take out the stimulator and integrated cuff

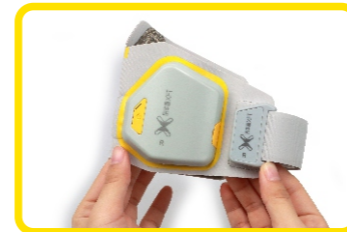

2/ Unfasten the snap buttons

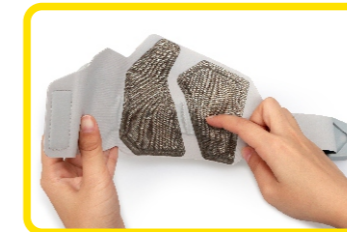

3/ Tear off the Velcro

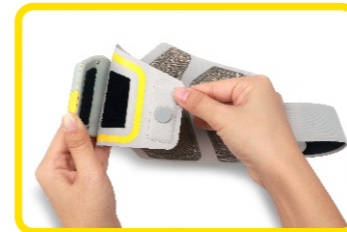

4/ Take the stimulator out of the integrated cuff



Shenzhen XFT Medical Limited

Room 203, Building 1, Biomedicine Innovations Industrial Park, #14 Jinhui Road, Pingshan New District, Shenzhen, China
E-mail: xft@xft.cn Tel.: 86-755-29888818

ShangHai International Holding Corp. GmbH (Europe)
Eiffestrasse 80 D-20537 Hamburg Germany.



www.xft-china.com

■ C80 M100 ■ C50 Y100 ■ M10 Y100 尺寸: 110X160MM



文件名	2001D 英文拆装图	料号	4990000026	版本	V 1.0	修改记录
材质	300克铜版纸过哑胶	单位	MM	日期	20180226	
核准		审核		制作		