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ART. 1007

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SOFT FIXATION CERVICAL COLLAR

Indications for use:

Cervical vertebra and soft tissue neck injuries, accidental or abrupt movement of head or neck, at postoperative stage.

- Limits the movement of the cervical spine, reducing pain syndromes
- Supports cervical spine in correct position
- Provides warming effect and soft massage
- Foam provides softness and comfort during usage
- Velcro closure for easy and tight adjustment

Application:

Before use please consult with your doctor.

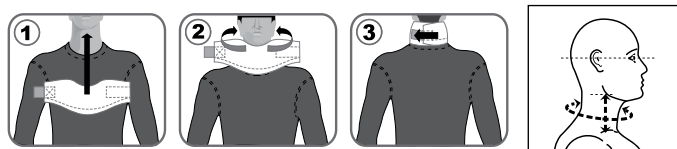
1. Select bandage size according to neck height and circumference in accordance with the table below.
2. Place the bandage at the front surface of your neck. Place the hollow under the chin.
3. Embrace your neck with the bandage, adjust length corresponding to neck size and fasten the bandage with the clasp on back side of the neck.
4. The doctor shall define individual treatment and terms of usage.

Contraindications:

Skin local diseases, individual intolerance of the raw material components.

Dimensions	Sizes		Max deviation
	I	II	
Neck height, cm	9,0	11,0	± 1,0 cm
Neck circumference, cm	28-39		± 1,0 cm

How to get it on:

Maintenance: Wash with mild detergents at temperature no higher +35°C. Don't rub or wring out during washing and rinsing. Dry up away from sources of heat. Don't iron. Don't bleach. (Don't use chlorine-containing bleaching detergents). Don't use dry-cleaning.

Precautions: Use the product for its intended purpose only. The schedule or time of wearing can be changed upon your doctor's advice only. It is not recommended to wear the product within 30 minutes after application of local warming ointments.

Contents: polyurethane foam – 88%, viscose – 10%, polyether – 2%.

Storage conditions: store in dry place at temperature from +5°C to +40°C. Keep away from direct sun rays.

Manufacturing date: see on packaging.

Size: see on packaging.

MedPack Swiss GmbH, Switzerland, Tramstrasse 16, 9442 Berneck,

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MedPack Swiss GmbH ist ein Mitglied der MedPack Swiss Group, Schweiz.