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ART. 4012

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## DIRECTIONS FOR USE

ART. 4012

UNIVERSAL SUPPORT  
(MEDICAL BINDER PREVENTIVE ELASTIC)

## Indications for use:

Binder is used to restore tonus at post surgical and post natal stage, prevents suture line disruption. It maintains abdominal muscles, provides moderate compression and reduces pendulous abdomen.

- Can be used to support abdominal muscles at post surgical and post natal stage
- Prevents suture line disruption and postoperative abdominal hernia
- Forms body contours and is invisible under clothes
- Breathable light fabrics of the bandage ensures maximal comfort during use.

## Application:

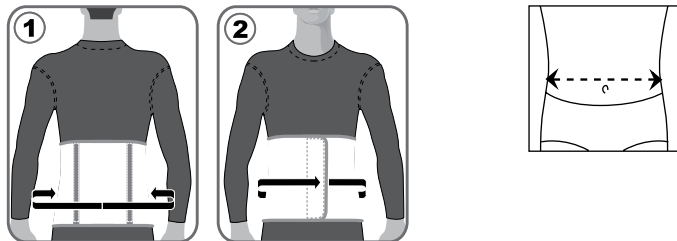
Before use please consult with your doctor. Required support size is chosen based on the waist circumference shown in the chart above. Support is recommended to wear directly on underwear. Individual treatment and terms are determined by the physician.

## Contraindications:

Inflammatory diseases of skin in the area of using the product, individual intolerance to materials the product is made of.

Dimensions	Sizes				Max deviation
	S	M	L	XL	
Waist circumference, cm	71-80	81-90	91-100	101-110	± 1,0 cm

## How to get it on:

## Maintenance:

Wash with mild detergents at temperature no higher +35°C. Don't rub or wring out during washing and rinsing. Dry up away from sources of heat. Don't iron. Don't bleach. (Don't use chlorine-containing bleaching detergents). Don't use dry-cleaning.

## Precautions:

Use the product for its intended purpose only. The schedule or time of wearing can be changed upon your doctor's advice only. It is not recommended to wear the product within 30 minutes after application of local warming ointments.

## Contents:

polyether - 56%, latex - 44%, plastic splints.

## Storage conditions:

store in dry place at temperature from +5°C to +40°C. Keep away from direct sun rays.

**Manufacturing date:** see on packaging.

**Size:** see on packaging.

**MedPack Swiss GmbH, Switzerland**, Tramstrasse 16, 9442 Berneck,

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MedPack Swiss GmbH ist ein Mitglied der MedPack Swiss Group, Schweiz.
Technomedica LLC, 24-A Berezhinskaya str, Dnepropetrovsk, Ukraine 49130,