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ART. 4507

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MATERNITY BELT (PRE- AND POSTNATAL)

Product description:

Typical features of bandage material are lightness, wearing qualities, and also free air and moisture permeability. Biologically inert fabric does not provoke allergic reactions.

Indications for use:

Prenatal bandage is specifically designed for expectant mothers to support the abdomen and pelvic organs in physiological rest position, favoring the normal position of the fetus. It helps alleviate discomfort and lower back pain due to pregnancy, unloading the lumbar spine. Wearing this bandage reduces the risk of excessive uterine activity, decreases stretching of the skin and amount of abdominal striae. The bandage is particularly helpful in cases of multiple gestation and polyhydramnios. Women with a history of previous spine traumas or disorders will also benefit while wearing the bandage.

- Reduces pains at lower back and hips
- Prevents stretch marks on the abdomen
- Helps to maintain correct posture and active lifestyle during pregnancy period
- Is invisible under clothing


Recommendations for use:

It is obligatory to consult your doctor before use. Size of the bandage is chosen according to the hip circumference measurements as shown in the table. The bandage should be worn over cotton underwear to avoid excessive sweating and a small chance of allergic reaction to materials of the bandage. The bandage regimen and duration of wearing are recommended on an individual basis by a medical doctor.

Contraindications:


Inflammatory skin disease in the area of brace application, individual intolerance to the brace materials.

Dimensions	Sizes			Max deviation
	S	M	L	
Hip circumference, cm	98-106	107-115	116-124	± 1,0 cm

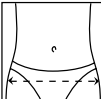
How to get it on:

schemes of bandage use before delivery

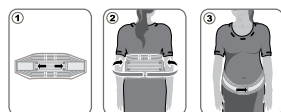schemes of bandage use after delivery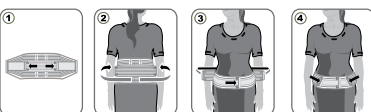

Maintenance:

Wash with mild detergents at temperature no higher +35°C. Don't rub or wring out during washing and rinsing. Dry up away from sources of heat. Don't iron. Don't bleach. (Don't use chlorine-containing bleaching detergents). Don't use dry-cleaning.

Precautions:

Use the product for its intended purpose only. The schedule or time of wearing can be changed upon your doctor's advice only. It is not recommended to wear the product within 30 minutes after application of local warming ointments.

Contents: polyether - 47%, latex - 41%, nylon - 12%.

Storage conditions: store in dry place at temperature from +5°C to +40°C. Keep away from direct sun rays.

Manufacturing date: see on packaging.

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MedPack Swiss GmbH, Switzerland, Tramstrasse 16, 9442 Berneck, phone: +41 71 747 11 95, fax: +41 71 747 11 91,
MedPack Swiss GmbH ist ein Mitglied der MedPack Swiss Group, Schweiz.