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ART. 9901

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ARM SLING (FACILITATED)

Indications for use:

- Arm fractures, ligament sprains, within postoperative stage
- Provides arm support and reduces pressure onto neck
 - Additional strap provides superb stabilization of the arm
 - Breathable fabric for comfortable use
 - Fits for both left and right arms

Application:

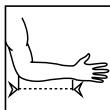
Before use please consult with your doctor.

Required sling size is chosen based on the arm length shown in the chart above. Use Velcro straps to adjust the best arm position. Individual treatment and terms are determined by the physician.

Contraindications:

Inflammatory skin disease in the area of brace application, individual intolerance to the brace materials.

Dimensions	Sizes			Max deviation
	S	M	L	
Forearm length, cm	21-27	28-33	34-40	± 1,0 cm

How to get it on:

for right hand

for left hand

Maintenance:

Wash with mild detergents at temperature no higher +35°C. Don't rub or wring out during washing and rinsing. Dry up away from sources of heat. Don't iron. Don't bleach. (Don't use chlorine-containing bleaching detergents). Don't use dry-cleaning.

Precautions: Use the product for its intended purpose only. The schedule or time of wearing can be changed upon your doctor's advice only. It is not recommended to wear the product within 30 minutes after application of local warming ointments.

Contents: cotton - 80%, polyether - 15%, nylon - 5%.

Storage conditions: store in dry place at temperature from +5°C to +40°C. Keep away from direct sun rays.

Manufacturing date: see on packaging.

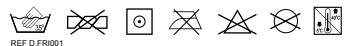
Size: see on packaging

MedPack Swiss GmbH, Switzerland, Tramstrasse 16, 9442 Berneck,

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MedPack Swiss GmbH ist ein Mitglied der MedPack Swiss Group, Schweiz.