## **Instructions**

Wash the Treat–Eezi before use. Place directly under the pressure sore or area at risk of developing a pressure sore. Best results occur when the sore is directly in contact with the product. If a garment or undergarment is worn, ensure it is as thin as possible and allows air circulation (e.g. A thin cotton shirt or thin cotton undergarment.) Do not place any other pad, bedding or cushions over the Treat–Eezi pad. Skin creams can be used in conjunctions with the Treat – Eezi pad.

Wash using a PH neutral detergent. Decontaminate using a Sodium Hypochlorite solution of 1000ppm in line with HSG9518 and BHTA guidelines. Ensure product is completely dry before patient use. Dry naturally or on a cool tumble setting.

## DO NOT IRON. DO NOT USE FABRIC SOFTENERS

<u>DISCLAIMER</u> considering individual circumstances may be appropriate for the use of these products standard medical care must be followed at all times. These products are not intended to replace and procedure or guidelines in place for the prevention of pressure wounds.

<u>WARANTY</u> this product is guaranteed to be free from defects in materials and workmanship under normal use for a period of one year.

www.danmedicasouth.co.uk