

Maxhim®

Intimate Essentials for Men



Maxhim® Vacuum Erection Pump
Instructions for Use

CONTENTS



- A** Pump cylinder x 1
- B** Ring ejector x 1
- C** Mounting cone x 1
- D** Lubricant x 1
- E** Constriction rings x 7
- F** Seal rings x 2
- G** Body shield x 1
- H** Battery powered pump x 1
- I** Manual pump x 1
- J** Vatra industrial AA batteries x 2


CARE GUIDELINES

Ensure all parts of the Maxhim Vacuum Erection Pump are clean and dry prior to use. We recommend trimming or full removal of pubic hair around the base of the penis prior to use, this will ensure a proper seal between the pump and the base of the penis.

The battery powered vacuum pump cannot be submerged or soaked in water, clean only using a damp cloth if necessary. All other parts may be washed with warm soapy water. Ensure you use a soap you know will not irritate your skin.

INSTRUCTIONS FOR USE

STEP 1.

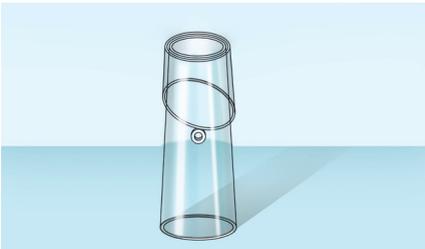

Place the pump cylinder (A) on a flat surface with the narrowest end facing upwards and place the ring ejector (B) over the pump cylinder with the slanted end facing downward.

STEP 2.



Insert the mounting cone (C) on the pump cylinder with the pointed end facing upwards.

The mounting cone should fit comfortably in the pump cylinder without force.

Now, lubricate the mounting cone lightly with the lubricant supplied (D).

STEP 3.



Select a constriction ring (E) from the 7 available.

We recommend you use the largest one that will hold your erection whilst also providing enough constriction to prevent blood leaving the penis


You should never wear a ring that feels too tight as you can cause bruising or damage to the penis. Slide the constriction ring down over the mounting cone until it sits on the pump cylinder. If you have any problem sliding the constriction ring over the mounting cone, you may need to add a little more lubricant.

STEP 4.




Remove the mounting cone and in its place put a seal ring (F). There are two different sized seal rings to choose from, select the one that your erect penis will easily fit through.

STEP 5.

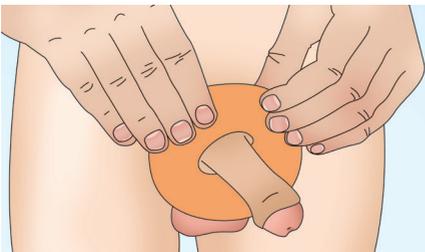

Place the body shield (G) over the penis and against the pelvis. At this stage, you should lubricate your penis to allow it to glide easily up the cylinder wall.



STEP 6.

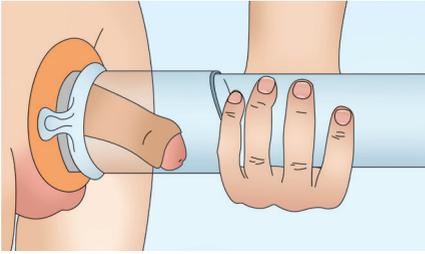


Place the pump cylinder over the penis. The end with the constriction ring on it should be closest to the body.

STEP 7.


Place the pump you wish to use on the end of the cylinder. You can either use the battery pump (H) or the manual (I).

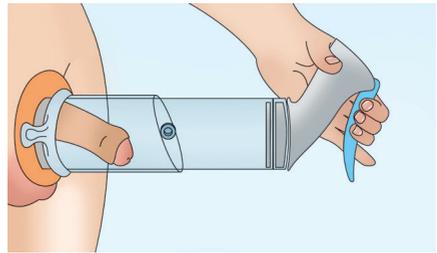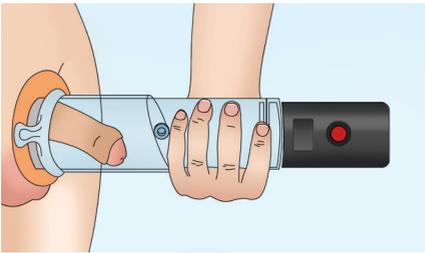

USING THE BATTERY PUMP

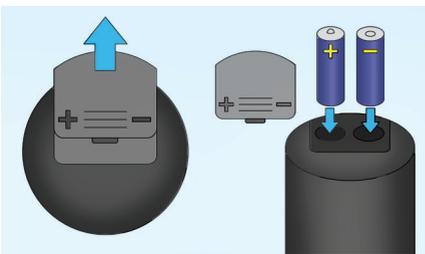


Insert the batteries into the battery pack as shown left, ensuring the polarity of the batteries corresponds with the positive (+) and negative (-) symbols on the cover.

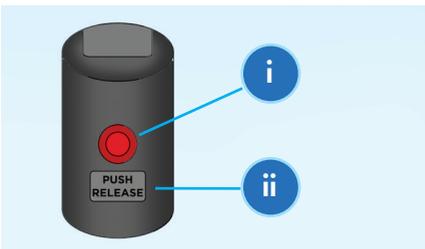

The battery pump has a power button (i) and a release valve (ii). Should you need to, you can release some of the pressure inside the pump cylinder by pressing the release valve.

STEP 8.

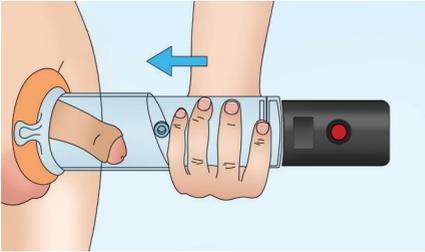

Press the red 'start' button whilst applying gentle force to push the entire unit towards the pelvis. The force applied should ensure there is an adequate seal and you should feel the suction as soon as the unit is turned on.

If you do not feel the pump working, it is likely there isn't an adequate seal. Try adding lubrication behind the body shield or between the pump cylinder and body shield, or both.

STEP 9.



Once an erection is achieved, twist the ring ejector to simultaneously break the suction seal and push the constriction ring off the pump cylinder and on to the base of the penis.

STEP 10.



When you are erect, remove the body shield before intercourse, leaving the constriction ring in place (this is what will help you maintain your erection).

STEP 11.



Remove the constriction ring after intercourse using the small, stretchy handles on either side.

IMPORTANT INFORMATION

- You should only ever apply the minimum amount of vacuum pressure necessary to achieve an erection. If you pump for too long you will possibly bruise your penis causing discomfort.
- The Maxhim Vacuum Erection Pump has been designed to help you achieve an erection in 20 seconds or less, please ensure you do not over pump. You should not be overly concerned if you notice some painless bruising or red spots on your penis. However, if you experience heavy bruising or pain from over pumping or pumping for too long, you should consult a physician if you are concerned and wait 5-6 days before using the Maxhim Vacuum Erection Pump again.
- Do not leave the constriction ring on for more than 30 minutes, doing so could cause permanent damage to your penis. You should also wait for at least 1 hour after use before using again.
- Do not use a constriction ring if you have decreased sensation or pain in the penis.
- You should ensure you have the strength to remove the constriction ring when required, or that you have someone with you to help if necessary.
- You should never use the Maxhim Vacuum Erection Pump when under the influence of alcohol or drugs that may reduce cognitive function. You must ensure you are capable of feeling pain, should it occur.

continued...

- Stop using immediately if pain occurs.
- Not suitable for those with Sickle Cell Disease.
- Do not use if you are taking large quantities of aspirin or blood thinners as they increase the likelihood of bruising and haematoma.
- Frequent use of constriction rings may cause bruising.
- Blood vessels may break during use.
- Consult your physician before use if you have recently had a penile implant removed.
- If you have any concerns at all, please consult a physician prior to use.



CAN WE HELP?

If you experience issues or have any questions at all, please do not hesitate to contact our expert customer services team.

Email: sales@StressNoMore.co.uk

Telephone: **01482 496 932** (Mon to Fri: 8am - 6pm & Sat: 9am - 5pm)