

WARNINGS AND PRECAUTIONS

The Maxhim® Adjustable Erection Ring is made of TPE thermoplastic rubber.

- Do not use the Maxhim® Adjustable Erection Ring for more than 30 minutes per use. Extended use of the Maxhim® Adjustable Erection Ring may cause permanent injury to the penis. Do not fall asleep wearing the Maxhim® Adjustable Erection Ring.
- Wait 60 minutes between uses of the Maxhim® Adjustable Erection Ring.
- Call your doctor if a problem occurs. Stop using the Maxhim® Adjustable Erection Ring if these problems persist.
- Do not use the Maxhim® Adjustable Erection Ring if you cannot feel pain in the penis.
- Do not use the Maxhim® Adjustable Erection Ring if you do not have adequate hand function to easily remove the device.
- Do not use alcohol or sedatives when using the Maxhim® Adjustable Erection Ring.
- Do not use the Maxhim® Adjustable Erection Ring as a method of contraception.
- Talk to your healthcare provider if any of the following factors apply:
 - If you have medical advice to avoid sexual activities.
 - If you have sickle cell anaemia, leukaemia, tumour of bone marrow or other conditions that change blood clotting and may result in bleeding or prolonged erections.
 - If you have an abnormally formed penis.

CAN WE HELP?

If you experience issues or have any questions at all, please do not hesitate to contact our expert customer services team.

Telephone: 01482 496931 / Email: sales@stressnomore.co.uk
(Mon to Fri: 8am – 6pm & Sat: 9am – 3:30pm GMT)

DIRECTIONS FOR USE:

Comfortably maintain a strong erection with the Maxhim® Adjustable Erection Ring. Made of rubber, simply place this easily adjustable erection ring around the base of the penis and tighten to reduce blood flow out of the penis. See the full guide inside for more information.

Check out the full Maxhim® range:

www.stressnomore.co.uk

Maxhim®
Intimate Essentials for Men



Manufactured for
Savantini Ltd
Savantini House,
Foster Street, HU8 8BT

IFU13228_0819

Maxhim®
Intimate Essentials for Men

ADJUSTABLE ERECTION RING

THANK YOU FOR CHOOSING MAXHIM®


Please read this information carefully before using the Maxhim® Adjustable Erection Ring. This device may cause injury if not used correctly.

Regular erections are the result of increasing blood flow into the penis. This blood is then stored by temporarily closing off the veins that drain the penis.


Some men are unable to sustain an erection because the veins of the penis stay open and prevent enough blood from being stored in the penis. This may be due to an existing health condition or as a result of prescription medication. Erection function can be restored with the use of the Maxhim® Adjustable Erection Ring.

The Maxhim® Adjustable Erection Ring is designed to improve erections by increasing the fullness and firmness of the penis. The Maxhim® Adjustable Erection Ring consists of an adjustable loop made of thermoplastic rubber that is placed around the base of the penis and tightened. This pressure reduces the flow of blood out of the penis.

The Maxhim® Adjustable Erection Ring may be tightened or loosened to fit.

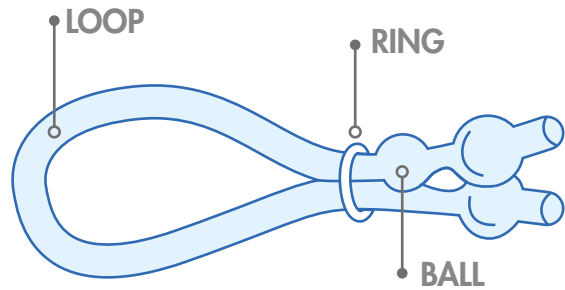


HOW TO USE

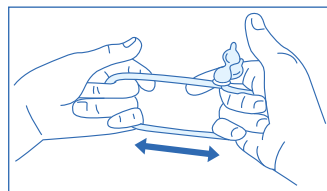


STEP 1.

Grasp the loop with both hands and stretch your Maxhim® Adjustable Erection Ring.

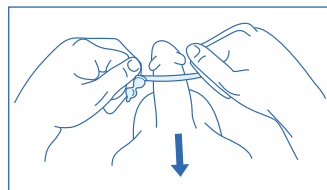


STEP 2.

Place the end of your penis through the stretched loop and slide the Maxhim® Adjustable Erection Ring over the shaft of your penis until it is around the base.

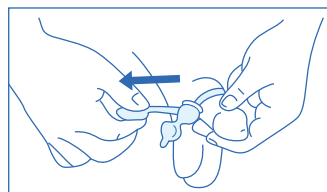


STEP 3.

Grasp the end of the tubing (without the ball) and pull away horizontally until the Maxhim® Adjustable Erection Ring is gently snug around the base of the penis.

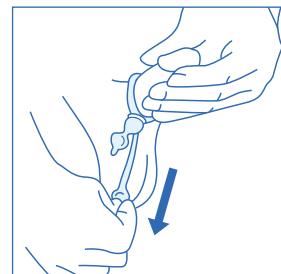

STEP 4.

For greater tension, grasp the tab on the end of the tubing (without the ball) and pull down. This should be tight enough to keep the blood from leaving the penis. However, too much tension may lead to discomfort and could cause harm.

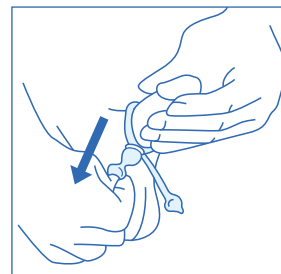

STEP 5.

To loosen the tension, pull on the end of the tubing with the ball. Once fully loose, the Maxhim® Adjustable Erection Ring can be removed by sliding it over the end of your penis.

TAKING CARE OF YOUR MAXHIM® ADJUSTABLE ERECTION RING

Your Maxhim® Adjustable Erection Ring is reusable. Clean the ring after use with warm water and mild soap. Store in a dry place.

Replace your Maxhim® Adjustable Erection Ring if the material becomes brittle or no longer has the tension to achieve the required stretch.