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Beppy Soft Comfort TAMPONS ^{UK}
 Comfortable lubricated tampons without string

Beppy WET tampons were developed for the woman who wants to be able to move freely during menstruation. The comfortable Beppy tampon without string is very suitable for use during sports, swimming, in the sauna, when travelling and for intercourse during menstruation. Insertion goes smoothly, thanks to the soft structure of the tampon and the lactic lubricating gel. Removal is easy using the removal loop. The special material of **Beppy WET** takes the shape of your body and absorbs menstrual fluid effectively.

- Instructions for insertion:**
- Wash your hands before inserting the tampon. Rinse them well with water, so that no traces of the soap remain on your hands.
 - Open the package.
 - Hold the tampon between your thumb and middle finger and place the index finger under the removal loop, the slanting part. Squeeze the tampon.
 - Adopt a comfortable position. The tampon can be inserted while standing, possibly with one leg raised above the other, or crouching. Keep the labia open with the other hand and, with the removal loop last, insert the tampon with the index finger till it reaches the back of the vagina, up against the cervix.
 - The **Beppy WET** tampon can stay in for up to 8 hours. Depending on the day of the period it may be necessary to replace the tampon earlier.

- Instructions for removal:**
- The **Beppy WET** is easy to remove using the handy removal loop. Wash your hands before removal and adopt a comfortable position.
 - Hook your index finger through the loop of the tampon and carefully pull the tampon out of the vagina. If this is not easy, crouch in a comfortable way or raise one leg up, so you can reach deeper. Press slightly and remove the tampon using your index and middle fingers as a tweezers, and gently pull the tampon out of the vagina. It is easier to remove the tampon after you have urinated, which helps you relax.
 - After use, do not deposit **Beppy WET** in the toilet.

How long to keep it in?
 Use Beppy tampons during normal to heavy blood loss (normal absorption capacity from 6-9 g). The length of time it can be worn depends on the amount of blood loss, and the day of the menstrual period. This is different for every woman. Guideline: every 2-4 hours during heavy flow, every 4-8 hours during the days of lighter flow. Change your tampon regularly. Don't forget to remove the last tampon before inserting another. Tampons may be worn overnight for up to 8 hours. **Always remove the last tampon at the end of your period.**

Please note:
 If you leave the tampon in the vagina for more than 8 hours, its safety and effectiveness is no longer guaranteed. Beppy WET can be used only once. Reusing them may cause infection. **The Beppy WET tampon is not a contraceptive.** Do not use tampons between your periods for regular discharge.

Do not use:
 - if your hymen is intact
 - If the use-by date on the pack has expired

IMPORTANT about Toxic Shock Syndrome (TSS):
 TSS is a rare but serious and sometimes fatal disease, which is associated with the use of menstrual tampons.
What are the symptoms of TSS? Early symptoms may begin suddenly and are similar to the flu, a sudden

fever, usually 39°C or higher, and vomiting, diarrhoea, fainting, dizziness or a rash that looks like sunburn. Early recognition and treatment is very important and can generally prevent serious illness. What should you do if you think you have TSS? If, during your period or shortly thereafter, you have any of the above symptoms, **REMOVE YOUR TAMPON AND CONSULT A DOCTOR IMMEDIATELY.** It is very important to tell your doctor that you have your period and have used tampons.
Who is at risk? TSS can occur in all women who use tampons during their periods. For women under 30, high risks have been reported.
 Reducing or avoiding the risk? To reduce the risk of TSS, use tampons with the minimum absorption required. To avoid or reduce the risk, do not use tampons or alternate the use of tampons with sanitary towels.

Consult a doctor if you want to use tampons again and have had symptoms of TSS in the past or have questions about TSS. As a general rule, it is necessary to visit your doctor, gynaecologist or A&E if you are cannot remove a tampon.

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