Stauringe / Spannungsringe

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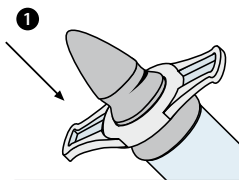
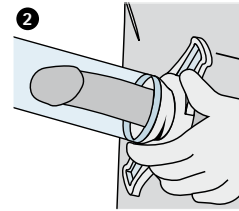
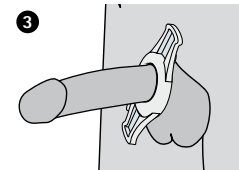
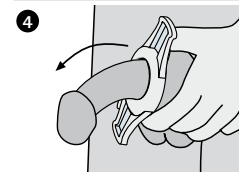
Constriction rings

for erectile dysfunction (ED) Vacuum erection device
for erectile dysfunction

ACTIVE *Erection System*^{MT} / MANUAL *Erection System*^{*}

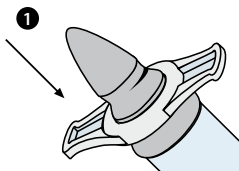
Using the constriction rings is key to the vacuum erection device working properly. The constriction rings come in the sizes 8 mm, 10 mm, 12,5 mm, 15 mm, 20 mm and 22 mm. When they use the device for the first time, men can find out which size ring best fits their penis: the one that produces a good erection (swelling effect) with the smallest amount of pressure and without any pain.

The constriction rings are intended for repeated use. They can be cleaned with soap and warm water. These constriction rings are subject to wear. They can tear after being used repeatedly, and need to be reordered.

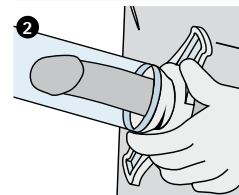
Note: **Never** wear the constriction ring on the penis for **longer than 30 minutes**. This will prevent the blood vessels from being damaged. Do not wear on the penis overnight.

Only use water-soluble lubricants together with this constriction ring.

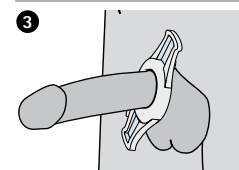
Contraindications: see the user leaflet for the vacuum erection device.

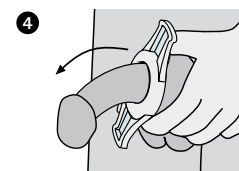
Mise en place de l'anneau de constriction
Sıkma halkasının geçirilmesi
Applicare l'anello costrittivo
Drukkring aanbrengen
سحب حلقة الاحتقان.

Passage de l'anneau de constriction du dispositif de stimulation de l'érection au pénis
Sıkma halkasını vakum ereksiyon cihazından penise geçirin
Trasferire l'anello costrittivo dalla pompa a vuoto sul pene
Drukkring vanaf de erectiepomp over de penis stropen
سحب حلقة الاحتقان من جهاز المساعدة على الانتصاب إلى القضيب.

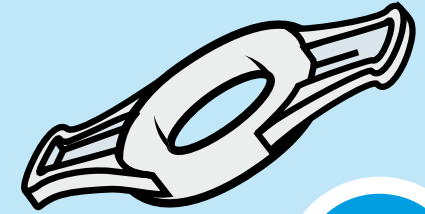
Position correcte de l'anneau de constriction pendant le rapport sexuel
Cinsel ilişkide sıkma halkasının doğru konumu
Posizione corretta dell'anello costrittivo durante il rapporto sessuale
Juiste positie van de drukkring tijdens seks
التوضيح الصحيح لحلقة الاحتقان أثناء عملية الجماع.

Retrait de l'anneau de constriction après le rapport sexuel
Cinsel ilişkiden sonra sıkma halkasının çıkarılması
Rimuovere l'anello costrittivo dopo il rapporto sessuale
Afstropen van de drukkring na seks
تجريد حلقة الاحتقان بعد انتهاء عملية الجماع.

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Holderinger / spenningsringer

for erektil dysfunksjonsterapi (ED)

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ACTIVE *Erection System*^{NT®} / MANUAL *Erection System*[®]

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